

Phoebe Putney Health System, Inc.

POLICY TITLE: Cleaning of Safety Goggles/Glasses Work Instructions

ENTITY: PPHS

Approved by: Infection Prevention

Effective Date: 3-14-20

Review Period: Annually

Review Date: 3-14-21

Contact Information: Infection Prevention Specialist

Scope:

Purpose: The purpose of this policy is to provide step by step guidance for the cleaning of safety Goggles/Glasses

Definitions: N/A

Policy:

Procedure:

1. Dirty glasses/goggles will be sent to CSP in Orange container.
2. Staff will view container, if any noted blood, no cleaning will be performed. The glasses will be placed into biohazard container to be destroyed.
3. If no blood or body fluid is noted, the glasses/goggles will be cleaned with an approval mild soap and water and dried.
4. The glass will then be disinfected with alcohol and placed in blue container for return to unit for use.